Kimetec GmbH
Gerlinger Str. 36-38
71254 Ditzingen • Germany

de CBC Venenstauer

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Purpose: Medical device to stop venous blood flow so that it is easier to locate and puncture a vein.

Contraindications/side effects:

None
Target group of patients:
No restrictions

Users: Only appropriately trained and qualified staff may apply the product.

Cleaning:

Do not apply mechanical cleaning processes. Wash the product with commercial washing agents that are gentle on fabrics at up to 60 °C (hand wash).

Disinfection:

Do not apply mechanical disinfection processes. We recommend disinfection with a liquid disinfectant (e.g. CIDEX® OPA Solution, Medos International Sàrl). Place the tourniquet in a container (non-metallic), cover it completely with the solution and leave it in the solution for at least 5 minutes. After the contact time, rinse it with plenty of fresh water (at least 3 rinsing rounds of at least 1 minute) and let it dry afterwards.

Disposal:

Please observe the local regulations.

Application:

Prior to using the product, always check it for external damage and proper working. Do not use the product in case of faults or faulty functions.

1. Close
2. Tie up
3. Slow release
4. Hold down, then open
5. Roll up

Any severe incident that occurs in connection with the product shall be reported to the manufacturer and the responsible national authority.

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