

# ORDER OF DRAW

FOR THE  
COMPLETE  
HANDLING  
INSTRUCTIONS

please refer to the  
Instructions for Use  
for each product at  
[www.gbo.com](http://www.gbo.com).

## VACUETTE®

### Blood Collection Tubes

based on CLSI GP41-ED7

1

BLOOD CULTURE

2

SODIUM CITRATE/CTAD\*

3

SERUM

with and without Separator

4

HEPARIN

with and without Separator

5

EDTA

with and without Separator

6

GLYCOLYTIC INHIBITOR

7

OTHER ADDITIVES



#### NOTE

Always follow general effective and safe working procedures and precautions during use and disposal of the products.

\* If a winged blood collection set is used, the first tube in the series will be under-filled. Therefore, if a Sodium Citrate specimen is drawn first, a discard tube (e.g. No Additive) is recommended to be drawn prior to this tube to ensure the proper additive-to-blood ratio. In addition, even though studies have shown that PT and aPTT tests are not affected if drawn first in a tube series, it is advisable to draw a second tube for other coagulation assays, since it is not known whether or not these tests will be affected.

This product information is addressed exclusively to healthcare professionals. Devices of Greiner Bio-One are to be used by properly trained healthcare professionals only in accordance with the relevant Instructions for Use (IFU). For a listing of indications, contraindications, precautions and warnings, please refer to the Instructions for Use which accompanies each product or is available for download from our website at [www.gbo.com/DownloadCenter](http://www.gbo.com/DownloadCenter). For more information contact your local Greiner Bio-One sales representative or visit our website.

All information is provided without guarantee despite careful processing. Any liability, warranty or guarantee of Greiner Bio-One GmbH is excluded. All rights, errors and changes are reserved. If not stated otherwise, Greiner Bio-One GmbH has all copyrights and/or other user-rights in this document. In particular to signs such as the mentioned word-picture-brands and logos. Any use, duplication or any other use of the rights of Greiner Bio-One GmbH is expressly prohibited. Media owner: Greiner Bio-One GmbH

[www.gbo.com](http://www.gbo.com)

Greiner Bio-One GmbH / Bad Haller Straße 32 / 4550 Kremsmünster / Austria  
PHONE +43 7583 6791-0 / FAX +43 7583 6318 / E-MAIL [office@gbo.com](mailto:office@gbo.com)

Poster Order of Draw en [rev.03, 04.2021]

  
**greiner**  
BIO-ONE